# CLINICARE PUMPSET

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso | Instrucciones de uso | Gebruiksaanwijzing

 **ARDO**

Ardo medical AG  
Gewerbestrasse 19  
6314 Unterägeri  
Switzerland  
T +41(0)41 754 70 70  
F +41(0)41 754 70 71  
info@ardo.ch  
www.ardomedical.com

 0123 Class IIa

 **ARDO**

99\_04\_208 / V02 / 06.2018

## 1. Uso previsto

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## 1. Intended use

The Clinicare Pumpset is to be used in conjunction with the Ardo Carum breastpumps, and it is intended for expressing breast milk.

en

## 2. Safety regulations

The pumpset is to be used only in a sterilized or cleaned condition, and it must be checked for damage after cleaning. The pumpset must not be used if the packaging is damaged or opened prior to first use. If expressing is painful, seek professional help. No contraindications are known. Warning: The Clinicare Pumpset comprises small parts and must therefore be kept out of the reach of small children. There is danger of suffocation.

## 3. Use

The Clinicare Pumpset is intended to be used by only one person. The maximum duration of use is 24 hours. The pumpset is intended for 8 pumping sessions within 24 hours. Disposable milk bottles in various sizes are additionally available.

## 4. Cleaning and hygiene

The Clinicare Pumpset is delivered ready for use. If the pumpset is used multiple times, it must be cleaned as soon as possible after each use. Individual parts that come into contact with breast milk (breast shell, lip valve, membrane) are disassembled, rinsed with cold, clean water, cleaned thoroughly with detergent solution and hot water and then rinsed with water. The parts are left to air-dry and then stored preferably in the Ardo Accessory Bag. The cover and tube do not need to be cleaned. They are disinfected by wiping, as needed.

## 5. Materials

Breast shell and cover, single tube connector (PP) / lip valve (silicone) / membrane (TPE) / tube (PVC)

Disposal: According to the local regulations.

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